Espagnol

Cojines KINÉRIS™

INSTALACIÓN/UTILIZACIÓN

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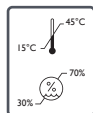
KINÉRIS™ Cushion

INSTALLATION/USE

The cushion is delivered ready inflated.

- Check cushion settings before first use and then monthly thereafter.
- Choose cushion size using the colour-coded guide, using the weight and size of the patient.
- Install the patient on the covered cushion and refer to the table below to adjust the air inside the cushion to optimum pressure level using the pressure gauge.

These values are indicative only and may vary with patient morphology.

Additional initial checks should be made as deemed fit.

- Circulate air in the network connecting the cells by carefully wobbling from left to right.
- Using the pressure gauge, recheck, and adjust if necessary.

Choice of the correct cushion height :

7 cm: Active Patient. Easy transfer to/from wheelchair

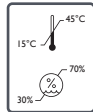
10 cm: Patients confined to wheelchair: Patient not normally moving from the wheelchair.

ENTRETIEN/DESINFECTION


- For cleaning purposes, it is best to use a microfibre cloth impregnated with detergent, disinfectant or other detergent product.
- Do not submerge the device in liquid to clean it.
- Do not sterilise.

PRECAUTIONS

- This pressure gauge is only for use with ASKLÉSANTÉ telescopic cell cushions.
- Protect the device from shock.
- Never inflate to more than 100 mmHg.
- Keep rubber parts away from sharp objects.
- Never expose the device to sources of heat or extreme low temperatures.

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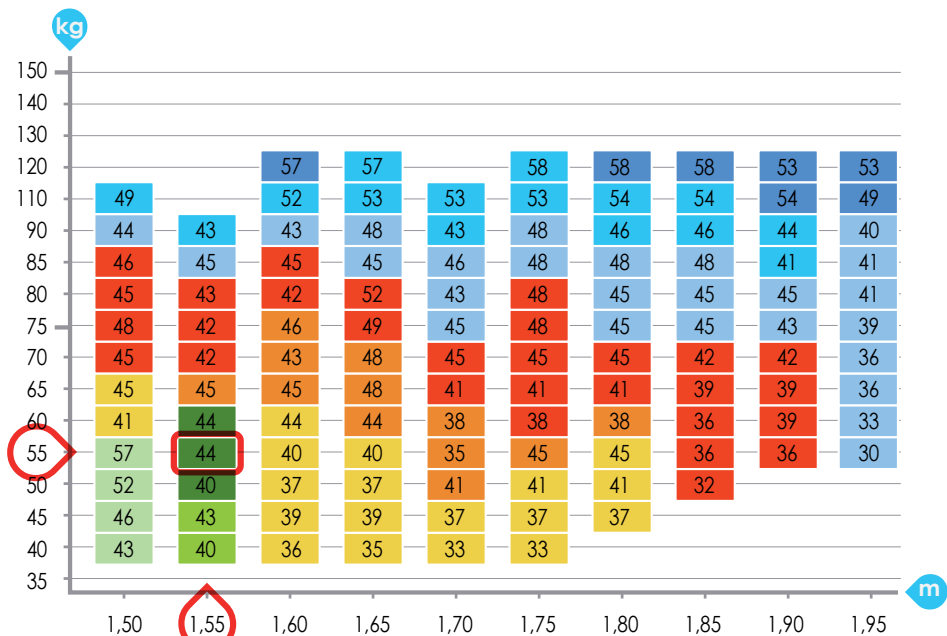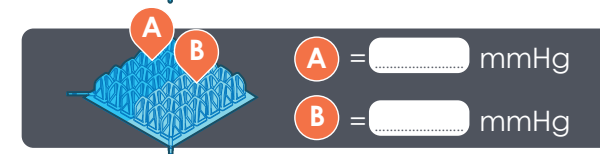
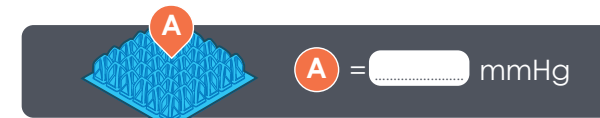
GB If the prescribed value is different :

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GB Ex. : Patient 1.55 m and 55 kg → 44 mmHg (see example in the table)

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